



DentCare

BQP-D15 3D Printer

& Q Slicer / Q Slicer PRO

Updating Guide



Q Slicer



Q Slicer PRO





BenQ AB DentCare Corporation

明基口腔醫材股份有限公司

Content


- Update FW
- Update Q Slicer
- Update Q Slicer PRO

BQP-D15 printer

Update FW


How to update BQP-D15 firmware version

1. Check the current FW version on your BQP-D15


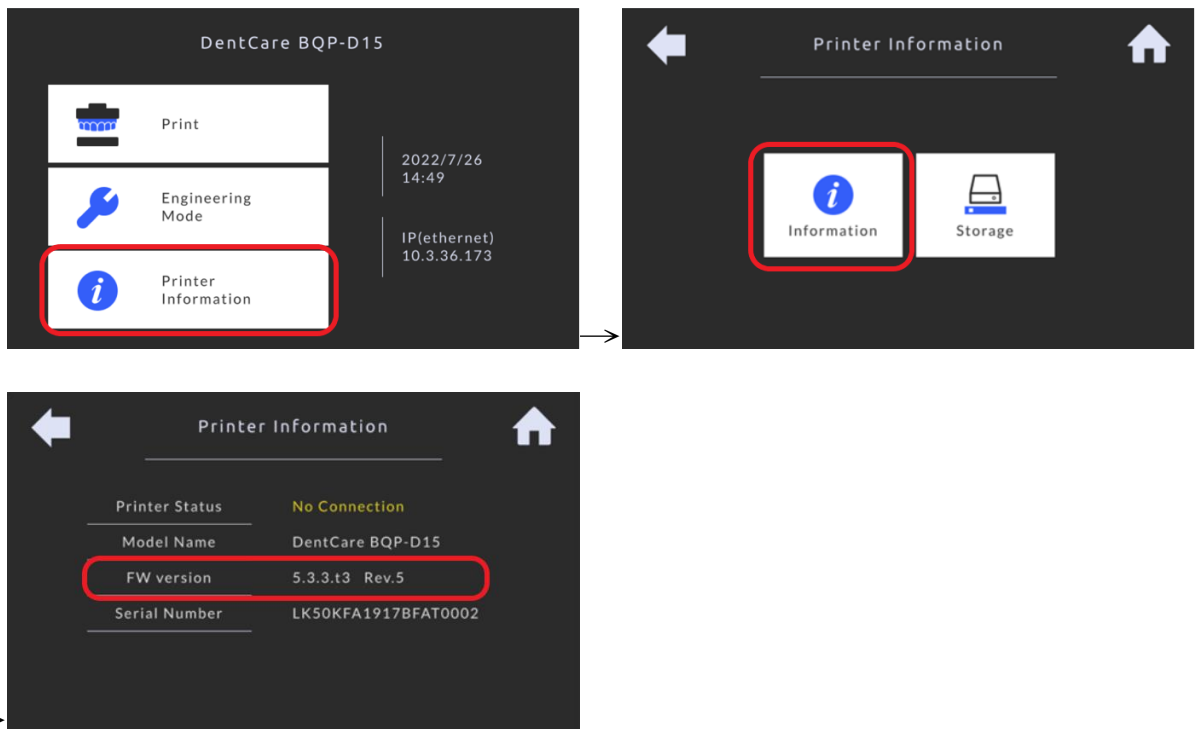


2. Download the latest FW version from DentCare website



<https://dentcarecore.com/downloads/en-US>

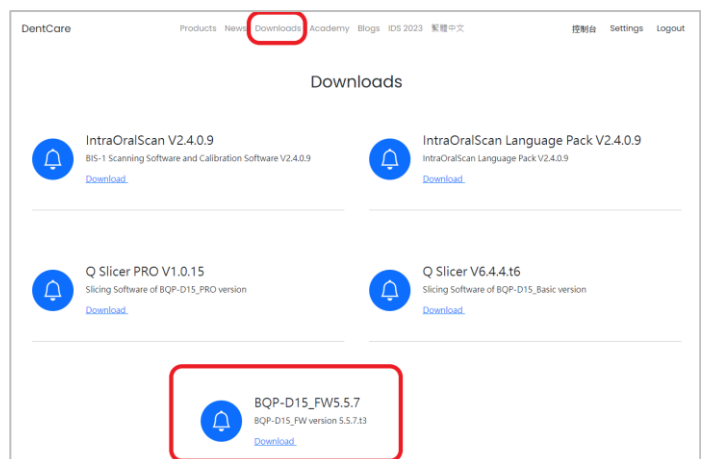

3. Connect BQP-D15 & laptop by RJ-45 cable

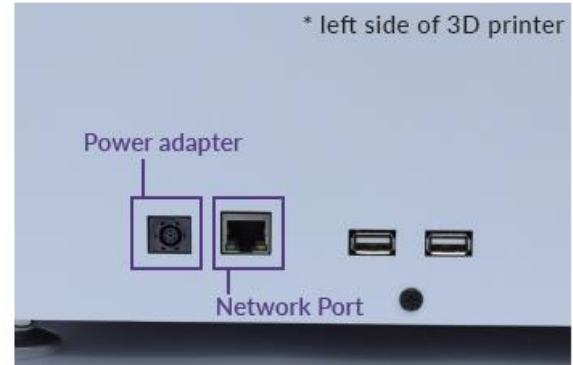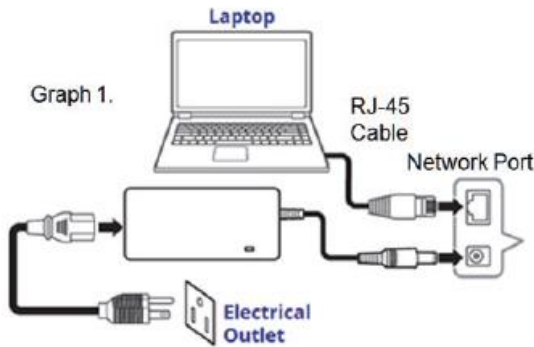


4. Enter your **printer IP + :8080** in address bar




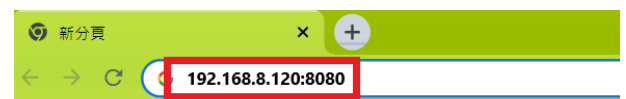
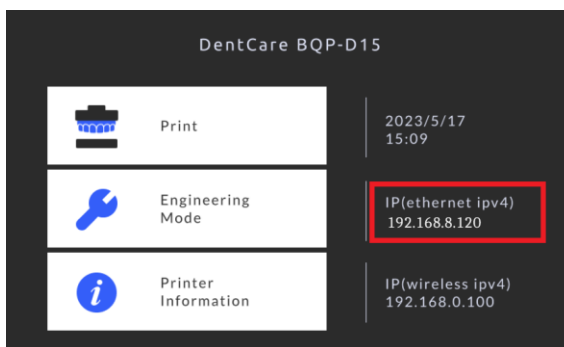


Enter
" printer **IP address** " + " :8080 "
in address bar



5. Go to the information page and choose the new FW version

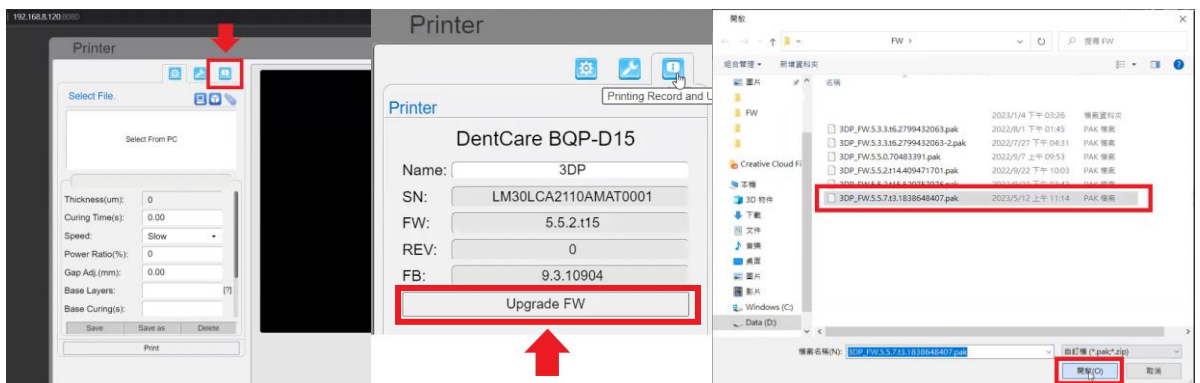

6. Please wait for the update to complete and refrain from performing any operations

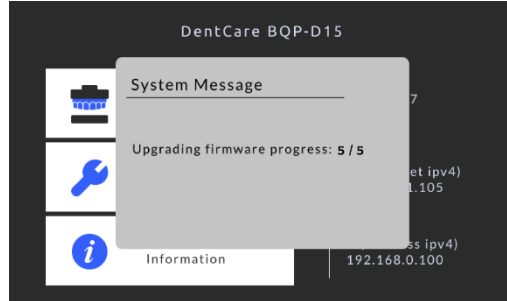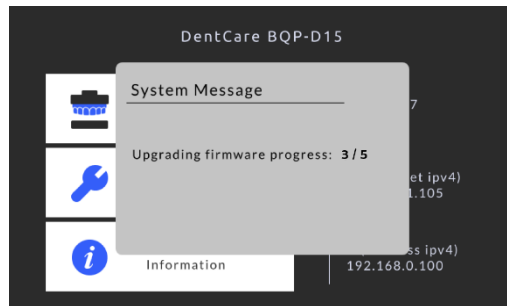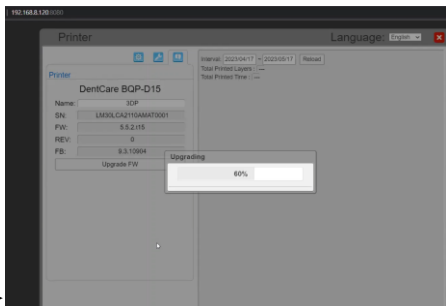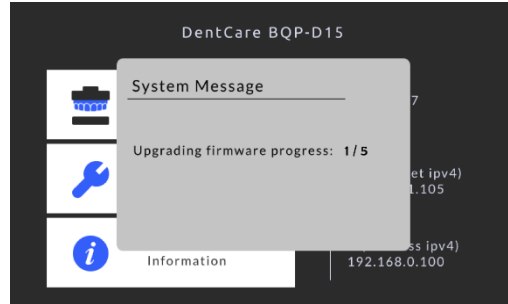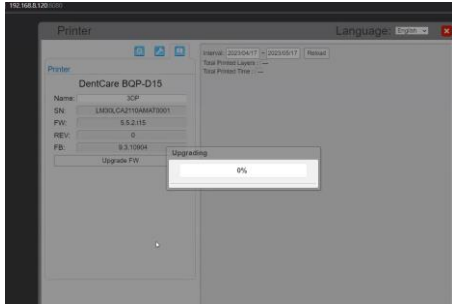

7. Your BQP-D15 will restart automatically, indicating that the update is complete

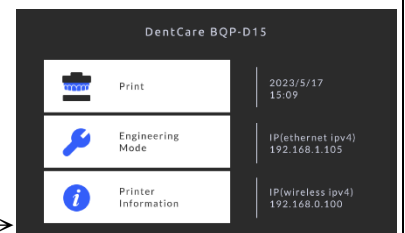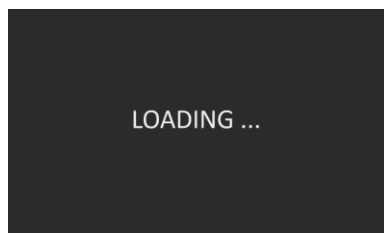


Q Slicer software

Update Q Slicer


Install a new version of Q Slicer

1. To check the current version of the Q Slicer software, please look for it on the top-left corner of the software interface.

 DentCare Q slicer Ver: 6.4.4.t6

2. Download the latest version from DentCare website


<https://dentcarecore.com/downloads/en-US>

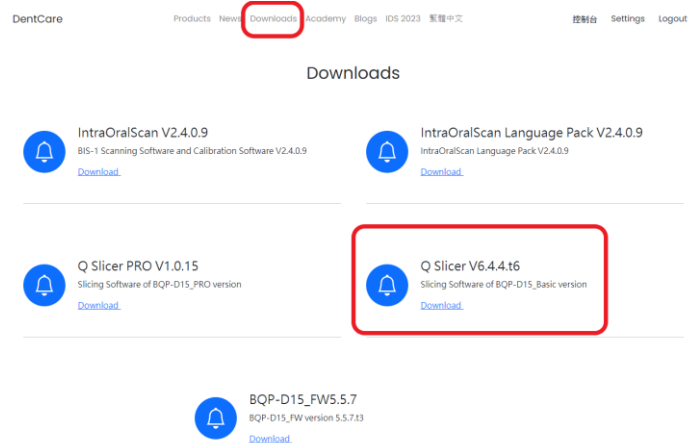


3. Delete the old version and unzip the downloaded file



4. Select "DentCare Q Slicer.exe" and export it to desktop

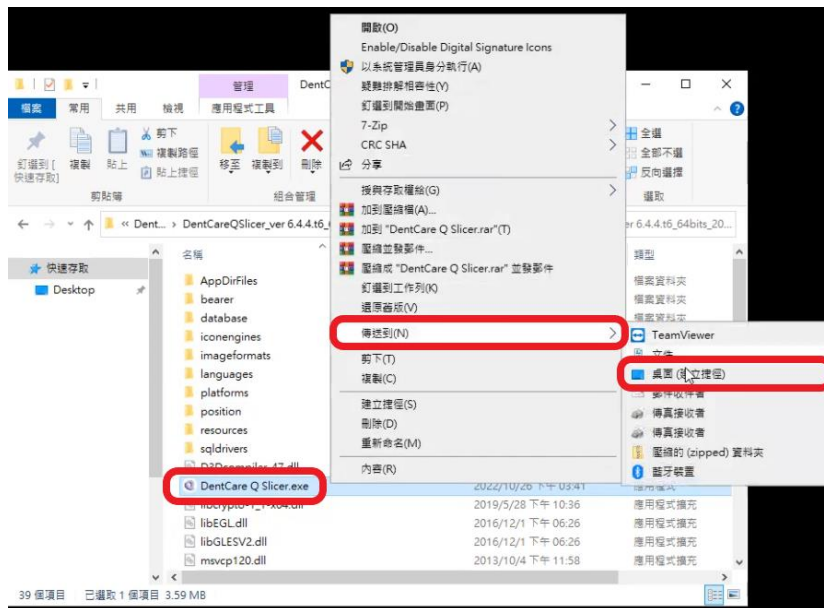

5. Open Q Slicer and choose " Q Slicer_Parameter "

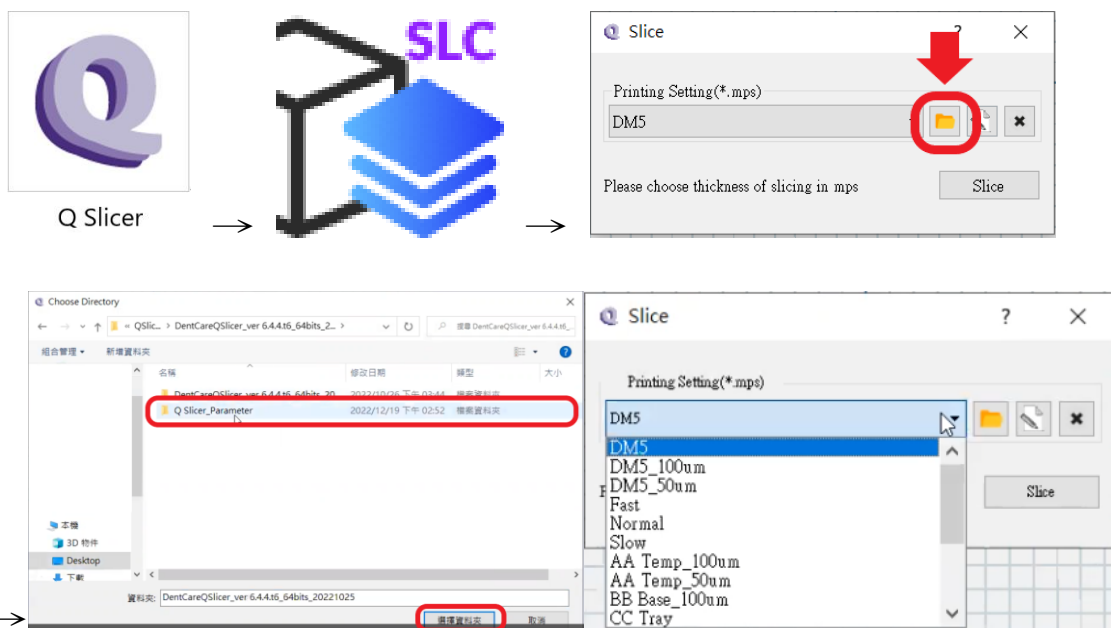


6. The installation is complete, and you can begin using it now.

Q Slicer PRO software

Update Q Slicer PRO


Install a new version of Q Slicer PRO

1. To check the current version of the Q Slicer PRO software, please look for it on the top-left corner of the software interface.


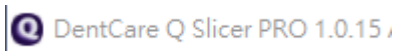



2. Download the latest version from DentCare website

<https://dentcarecore.com/downloads/en-US>

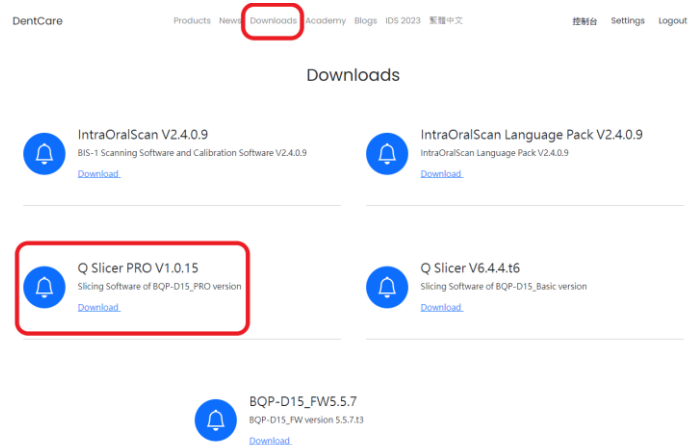

3. Double-click to install the latest version

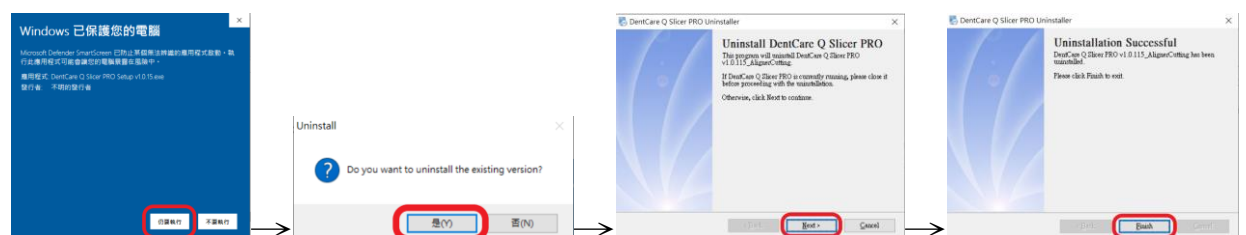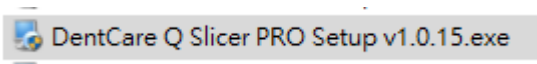


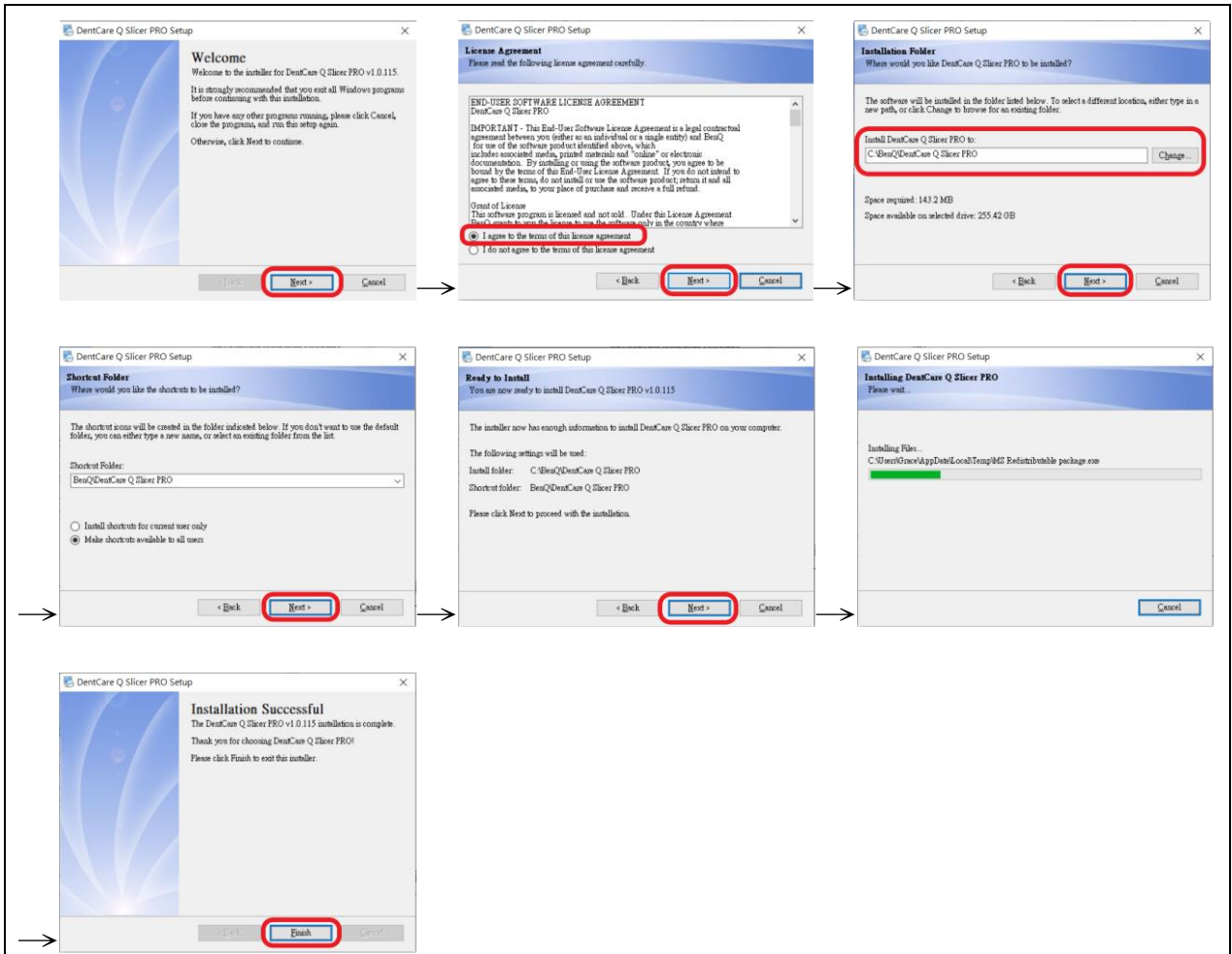

4. The installation is complete, and you can begin using it now



Q Slicer PRO

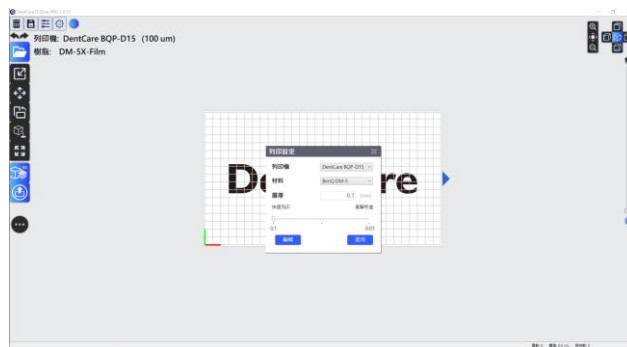



BenQ AB DentCare Corporation

明基口腔醫材股份有限公司

Support

Please visit our YouTube channel for more detailed operation steps:

<https://www.youtube.com/channel/UCB-KiIG9-bINsZQS5iOV55Q>

